

W TROUBLESHOOTING

Pairing BEYOND to iPhone

WIDEX BEYOND™
LIFE WITHOUT LIMITS

TROUBLESHOOTING SUGGESTIONS IF YOU ARE EXPERIENCING DIFFICULTY PAIRING:

- 1 Ensure that Bluetooth on the iPhone® is turned on and the hearing aids are close to the iPhone.
- 2 Verify that the hearing aids have fresh batteries. Wait 2 minutes after pulling the sticker off the battery before putting the battery in the hearing aid.
- 3 Open and close the battery doors of the hearing aids. You must complete pairing within 3 minutes of closing the battery doors.
- 4 Enable and disable Airplane Mode.
- 5 Verify that the iPhone is a compatible device. WIDEX BEYOND and the WIDEX BEYOND App are compatible with the following devices or newer: iPhone 6s, iPhone 6s Plus, iPhone SE, iPhone 6, iPhone 6 Plus, iPhone 5s, iPhone 5c, iPhone 5, iPad Air® 2, iPad Air, iPad (4th generation), iPad mini™3, iPad mini2, iPad mini with Retina display, iPod touch® 6th generation using iOS 9. WIDEX BEYOND App is optimized for iPhone 5S and newer iPhones.

FOR ADDITIONAL TROUBLESHOOTING, FOLLOW THE ADDITIONAL 4 STEPS IN THIS ORDER:

- 1 Unpair the hearing aids by going to Settings > General > Accessibility > Hearing Aids. Select your hearing aids and select “Forget This Device”.
- 2 Close the BEYOND App if open.
- 3 Force restart the iPhone that you are trying to pair.
- 4 Open and close the BEYOND battery door(s) and start the connection process again.

TROUBLESHOOTING SUGGESTIONS IF YOU ARE EXPERIENCING NO SOUND STREAMING OR POOR SOUND STREAMING:

- 1 Verify that the hearing aids are actively paired to the iPhone.
- 2 Verify that Bluetooth is turned on.
- 3 Verify that the Audio Routing is set to “Always Hearing Aids” or “Automatic” by going to Settings > General > Accessibility > Hearing Devices > Audio Routing.
- 4 Replace the batteries in the hearing aids with fresh batteries. Wait 2 minutes after pulling the sticker off the battery before putting the battery in the hearing aid.
- 5 Turn wifi off on the phone.



Apple, the Apple logo, iPhone, iPad, and iPod touch are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc.

GET CONNECTED. GO BEYOND.

WIDEX®

DRW268/1216