

PLEASE READ

Important Information regarding use of the iP5 Portable Programmer

With EPROM v.4.5

EPROM v. 4.5, or later, is required when using the iP5 programmer to program and fine tune the mind440 family of hearing aids.

mind440 Pre-Conditions

- Press **[==>]** in mode window to access **[AGE]** for the pediatric rationale. Apply the pediatric rationale if the client is under 10 years of age.

mind440 Program Options

- The default program setting **[PRG]** is the Master Program. The mind440 may have a maximum of four programs in addition to the Master Program. These programs may be acoustic or telecoil. The CIC and m models may have two acoustic programs in addition to the Master Program. The Master program provides the best overall performance in all environments. Each of the other available programs has a special focus. When programming with the iP5 the Master program is always the first program. Other programs may be added, changed or removed using the **[up or down]** arrow key corresponding to the desired program located below the main display.
- To choose SmartSpeak or beep tone signaling: Press **[==>]** in mode window. Press either **[up or down]** arrow key below the main display to choose **SPEAK** or **BEEP**. Press **[back]** to return to the Program Options mode. Note: The level of the SmartSpeak messages, the language, and the choice of speaker (male or female) can be *adjusted only in Compass*.
- **[RC4]** will be displayed in the mode window when programming a remote control compatible instrument. Press the **[RC4]** key to

access remote control matching. Press **[STRT]** in the first matching screen. In the next RC4 matching screen you must select the proper remote control model **[R4-1 or R4-2]**; separate or united (only available with R4-2) matching; light on/off; and alarm on/off. Press **[NEXT]** in the mode window. Three RC4 matching codes will be displayed. These codes must be entered in the RC4. Once the codes have been entered in the RC4 * the result of the matching process is shown. If **[MATCH OK]** is displayed the matching process has been successful. Potential matching errors include: choosing the RC4-1 when matching was attempted with an RC4-2 or choosing the RC4-2 when matching was attempted with an RC4-1. To return to the Program Selection screen hit **[BACK]** in the mode screen.

- * *The remote control is put into the programming mode by removing the batteries, immediately them and holding down the center and upper left keys simultaneously. The RC4 display will read **PRO**. Press the center key of the RC4 until the display reads **c1 (code 1)**. Enter code one displayed on the iP5 screen by using the **[up/down]** or **[2/3]** buttons. Press the center key again and enter **c2** (code 2). Press the center again and enter **c3** (code 3). Finally press the center key until the display reads **end**. The RC4 will automatically turn off within a few seconds. The settings and options are programmed and saved in the RC4.*
- * *If any changes to the number of programs or RC4 options are made, the remote control must be re-matched.*

Test Mode

mind440 self test:

- Turn programmer off by pressing On R and On L simultaneously.
- Press the appropriate On key while simultaneously pressing the bottom right menu key. SELF TEST is displayed.
- Press the menu key next to **[TEST]** to initiate the self test.

Additional Notes:

1. When using the iP5 programmer for mind440 fittings, the following features cannot be accessed or fine tuned:

- DAI programs
- Sound Diary
- In Situ RECD
- Zen programs/Smart Toggle
- QuickFit
- Clear Band offset